

**SWANSEA MASONIC HALL COMPANY Ltd.**

**LPOW**

**Non-Contact Infrared Body Thermometer**

**General Information**

The Hand Held Non-Contact Infrared Thermometer provided for use in SMH is designed for monitoring Human Body Temperature. The provided devices have been set up to read body temperature and therefore please do not adjust or change the settings. The device measures the temperature in Centigrade in addition to a traffic light system. When used the digital temperature will be displayed on the screen, additionally the relevant colour will be displayed, Green all good, Yellow doubtful, Red No entry. The World Health Organisation states that the normal range for Human body temperature is 36.5 to 37.5 degrees Centigrade.

**Guidance for Use**

1. Hold the device at arm's length approximately 1.5 to 5 cm away from the persons forehead.
2. Squeeze the trigger and hold until the reading and colour is displayed.
3. Green within range allows entry.
4. Yellow request person to stand aside for 5 mins and retake temperature. No changes on second reading refuse entry.
5. Red person is refused entry and informed of a raised temperature.
6. Record on attendance list.

20<sup>th</sup> November 2020.